

## ASEPTI ACTIVE LIQUID

### Hospital grade disinfectant

ARTG 340993

**Asepti Active Liquid** is a hospital grade disinfectant approved for single step cleaning and disinfection of dirty surfaces.

**Asepti Active Liquid** is a broad spectrum bactericide effective against VRE & MRSA and has virucidal efficacy against Human Coronavirus, COVID-19 [SARS-CoV-2] and Influenza A.



## DISINFECTANT CLEANER

**Asepti Active Liquid** contains a combination of quaternary biocides - Benzalkonium Chloride & Dialkyldimethylammonium chloride - designed to provide optimal efficacy against QAC resistant organisms.

CONDITION	EXAMPLE	CONTACT TIME
Broad Spectrum bactericidal efficacy including	<i>Pseudomonas aeruginosa</i> ; <i>Proteus vulgaris</i> <i>Escherichia coli</i> ; <i>Staphylococcus aureus</i>	8 minutes in dirty conditions
Bacterial Outbreak event Antibiotic Resistant bacteria	Vancomycin Resistant; <i>Enterococcus faecalis</i> - VRE; Methicillin Resistant; <i>Staphylococcus aureus</i> - MRSA	5 minutes in dirty conditions
Viral Outbreak event	Human Coronavirus COVID-19 [SARS-CoV-2] severe acute respiratory syndrome coronavirus 2; Influenza A	5 minutes in clean conditions 10 minutes in dirty conditions

**Asepti Active Liquid** is available in a handy ready to use 500mL bottle with flip top lid. The ready to use format minimises risk of cross contamination and incorrect dilution or mixing of chemistry.

**Asepti Active Liquid** is low foaming and offers a streak free finish.

**Asepti Active Liquid** is compatible with **Asepti Active Wipes**.

Use **Asepti Dry Wipes** (low lint, all purpose towels) as a companion item with **Asepti Active Liquid**.



Asepti Dry Wipes

## DIRECTIONS FOR USE .....

- ▲ Wear gloves and additional PPE where required.
- ▲ Apply **Asepti Active Liquid** on to the surface or via a dry clean wipe.
- ▲ Apply to surface in a normal wiping motion.
- ▲ Dispose of wipe after use and always wash hands.
- ▲ Do not dispose of wipes in toilets.

## SAFETY INFORMATION .....

Do not mix with any other products. For further information and safety instructions see product label and Safety Data Sheet.

Product Code	Product Description	Pack Size
7100752	Asepti Active Liquid	12 x 500mL
99001127	Asepti Dry Wipes	12 x 8 wipes
HC-00023	Wall Chart - Asepti Active Liquid & Wipes	Each

PLEASE SPEAK TO YOUR HEALTHCARE MANAGER OR CALL **1800 022 002** FOR FURTHER INFORMATION